

PRODUCT FOCUS:

Enuresis Sensor Kit



TASK Community Care | Product Focus

Enuresis Sensor Kit



Overview

This product is designed to inform caregiver when incontinence needs to be addressed. The enuresis sensor kit is easy to install as it is simply placed under a sheet on a bed or on a chair.

How the System Work

The sensor pad is connected via a cable to the alarm device which can be easily attached to the frame of the bed or chair. The device is programmed to sound an alert when wetness is detected. The alarm delay time is adjustable according to the user's need. There are different alarm sounds for selection.

It can be used as a standalone device or installed as part of our Carelink integrated Telecare system.

This means an onsite carer or staff in our 24hr-monitoring centre can take appropriate action to ensure a comfortable resolution to the individual's needs.

Benefits

Reduces the discomfort of an individual, allowing for prompt notification of detection of wetness.

Sensor Pad Lifetime and Size

Each enuresis sensor pad has a lifetime of 30 days. Individual replacement pads are available. Pads should be disposed after detection of wetness. They are not washable.

2 Sizes Available: 380mm x 305mm (for chairs) or 762mm x 102mm (for beds)

Audible Alert Device Features and Warranty

Colour: White Size: 134mm x 87mm x 48mm

Power Source: Operated with 3 AAA batteries (device has a low battery indicator).

Time of Delay Message: Adjustable from 0, 1 and 2 seconds delay

Alarm Tone Options: 3 Multiple (Low, Mid, High volume)

Warranty: 1 Year

Part of the TASK CareLink Falls and Memory Support Packages, for service users who have a substantial or critical need. Please contact us for further details on this or other TASK CareLink Call Packages

For more information on the Enuresis Sensor Kit, please contact us at:

TASK Community Care
Unit 1, Block 3
CityNorth Business Park
Stamullen
County Meath
IRELAND

info@taskltd.ie
Telephone: +353 (0)1 8435889
www.taskltd.ie

TASK Community Care
The Innovation Centre
Northern Ireland Science Park
Queen's Road
BELFAST
BT3 9DT

info@taskltd.com
Telephone: 0845 304 5535
www.taskltd.com

© Copyright TASK Community Care 2015. Product catalogue printed 02..2015. Please note: Actual product may differ slightly from image shown.